DE

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GB

INSTRUCTIONS FOR USE

DESCRIPTION:
All catheters comprise a 2-lumen shaft with proximal funnel, inflation valve and distal filling balloon.
Balloon fill volumes are in ml, and the shaft sizes are in French (Fr.), Charrière (Ch.) and are indicated on the funnel of each individual catheter and on the individual pack label and outer carton. The distal tip type is indicated on the funnel of each individual pack label and/or outer carton.
The bladder catheter is made from 100% silicone.

Sizes: 6 to 24 Fr./Ch.
Lengths: 31 cm for the sizes 6 to 10 Fr./Ch.
40 cm for the sizes 12 to 24 Fr./Ch.

Size (Fr./Ch)	Bladder volume	REF	PZN
6	3,0 ml	23.000.14.206	04365715
8	3,0 ml	23.000.14.208	04365721
10	3,0 ml	23.000.14.210	04365767
12	5-10 ml	23.000.14.212	04365773
14	5-10 ml	23.000.14.214	04365796
16	5-10 ml	23.000.14.216	04365833
18	5-10 ml	23.000.14.218	04365856
20	5-10 ml	23.000.14.220	04365862
22	5-10 ml	23.000.14.222	04365968
24	5-10 ml	23.000.14.224	04366005
16	30 ml	23.000.24.216	04366028
18	30 ml	23.000.24.218	04366034
20	30 ml	23.000.24.220	04366040
22	30 ml	23.000.24.222	04366057
24	30 ml	23.000.24.224	04366175

INDICATIONS:
These catheters are indicated for routine drainage of the bladder or for routine post-operative drainage.
READ ALL INSTRUCTIONS CAREFULLY BEFORE USE.

PREPARATION FOR USE:
Remove catheter from its sterile pouch using an aseptic technique.
Visually inspect the catheter for any mechanical damage. If a trial Inflation of the balloon is carried out, ensure the balloon is empty prior to insertion.
Only sterile water should be used to fill the balloon.

DIRECTIONS FOR USE:
Lubricate the catheter using only a water based lubricant or gel (with or without local anesthetic as required). Using a sterile technique as indicated in published literature; insert the catheter fully into the urethra ensuring that the balloon is beyond the bladder neck. All angled catheters tips correspond to the direction of the catheter valve.
A Luer syringe may be used to inflate the balloon via the non-return valve.
Inflate the balloon with sterile water. The volume of the balloon is indicated on the funnel of the catheter on/and on the label of the packaging. Then connect the catheter with the urine collection system.

WARNING: DO NOT OVER INFLATE.
Ensure function by checking for urine flow from the drainage funnel.
To deflate the balloon prior to removal of the catheter, insert a Luer syringe as appropriate.
Aspirate the syringe gently to remove the inflation fluid. DO NOT use excessive aspiration on the syringe during deflation as this may cause a vacuum collapse of the inflation lumen, which may impair aspiration of inflation fluid. Maximum recommended indwelling time is 30 days.
The inflation balloon holds the catheter in the bladder. If the balloon loses fluid, e.g. by diffusion of the filling fluid through the balloon membrane, the catheter can, with time, slide out of the bladder inadvertently. In order to prevent this danger, the inflation liquid should be withdrawn from the catheter at least once every 2 weeks and the balloon should then be immediately reinflated with the stated nominal filling volume. The patient should be routinely monitored in accordance with accepted procedures and the catheter removed after a suitable interval, not longer than 30 days, as determined by a physician or other suitably qualified personnel.

POTENTIAL COMPLICATIONS:
If difficulty is encountered aspirating the balloon with a syringe, a rare and infrequent reported event, the branch of the catheter with the valve should be cut with sharp scissors at the bifurcation or the balloon ruptured according to established procedures reported in medical literature. Should it be necessary to rupture the balloon, care must be taken to remove all fragments from the patient's bladder. Incorrectly positioned catheters can cause urethral damage if the balloon is inflated within the urethra. Irritation of the urethral mucosa, blockage of the catheter due to encrustation and catheterinduced infections are documented complications with some patients.

CAUTION:
For urological use only.
Use only a water based lubricant.
Do not use lubricants with a petroleum base.
Always inflate the balloon with sterile water.
Never clamp the catheter; a catheter plug or valve may be used if required.
Never use a syringe to puncture the catheter shaft for urine sampling. There is a danger of needle stick injury and the catheter may be compromised.

PRESENTATION :
Each product is supplied in an individual sterile pouch.
The product is guaranteed sterile unless the individual packaging has been opened or damaged. The device is for single use only. Reuse risks spreading infectious bacteria. Sterilization and washing (for reuse) may impair performance of the device and causes harm to the patient.

STORAGE:
Do not store our products in extreme temperatures or wet conditions.

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