Adaptor nutrisafe2-ENFit

REF : 368.72


Adaptor nutrisafe2-ENFit

English

Nutrisafe2 - ENFit adaptor
Ref: 368.72

DESCRIPTION

The Nutrisafe2-ENFit adaptor is indicated for enteral use only. It enables the connection between an administration set equipped with a female ENFit connector at its distal end and a Nutrisafe2 feeding tube equipped with a male Nutrisafe2 connector at its proximal end.

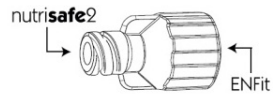Priming volume : 0.10 ml

SETTING UP

1. Assemble all the equipment required:
The feeding tube from the Nutrisafe2 range
The administration set with the ENFit connector
The Nutrisafe2 - ENFit adaptor

2. Placement of the adaptor:

First, screw the adaptor on the ENFit side to the administration set equipped with a female ENFit connector. Secondly, screw the adaptor on the Nutrisafe 2 side to the feeding tube equipped with a male Nutrisafe 2 connector.

3. Control :

Confirm that adaptor is connected to an enteral port and not an IV set. Check that the adaptor is in its proper place and screwed on each side before use.

The rules of good hygiene should be followed at each step of the adaptor placement.

PRECAUTION FOR USE

- Only use this adaptor in case of Nutrisafe2 and ENFit medical device use.
- Do not maintain the adaptor on the feeding tube, remove it consistently with administration sets. It can be used 24 hours maximum (single-use device).
- Re-use of this device may change its mechanical and biological features and may cause device failure, allergic reactions or bacterial infections.
- For disposal, place the contaminated items in the appropriate receptacle.
- This product does not contain natural rubber latex or dry rubber latex.

WARNINGS

This device incorporates connectors which are designed to reduce the likelihood of tubing misconnection. However, the potential to misconnect this device with medical devices of other healthcare applications still exists, including intravascular devices, neuraxial devices, hypodermic applications, breathing systems and driving gas devices, urinary devices and limb cuff inflation devices.

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