

HOSPITAL CARE

Tourniquet

Tourniquets for Venous Access

Product Information



Tourniquets are an essential tool to prepare patient for venous access

The design of Terumo provides comfort for health care professionals as well for patients and facilitates fast and efficient interventions by means of:

- One-hand operation
- Quick-release lock
- Safety snap closure

| Description | Raw Materials |
|---------------------|---|
| Plastic housing | Polycarbonate (PC) |
| Other Plastic Parts | Polyoxymethylen (POM) |
| Elastic | Polyester and Lycra®-phthalate free |
| Colours of Ribbon | Does NOT contain chemical substances which are prohibited in textiles |

Does not contain components made of DEHP or natural rubber latex.

Designed for easy and intuitive handling

Product Details

Cleaning

Do not apply mechanical cleaning processes. Wash the product with commercial washing agents that are gentle on fabrics at up to 60°C (hand wash).

Disinfection

Do not apply mechanical disinfection processes. We recommend disinfection with a liquid disinfectant (e.g. CIDEX® OPA Solution, Medos International Sàrl). Place the tourniquet in a container (non-metallic), cover it completely with the solution and leave it in the solution for at least 5 minutes. After the contact time, rinse it with plenty of fresh water (at least 3 rinsing rounds of at least 1 minute) and let it dry afterwards.

Disposal

Please observe the local regulations.

Product Range

| Product Code | Ribbon Width | Colour/Decoration | Supply Unit |
|--------------|--------------|-------------------|-------------|
| XX+TC | 25.0 mm | Multi-Colour | 100 pcs |
| XX+TCB1 | 25.0 mm | Dinosaur | 100 pcs |
| XX+TCB2 | 12.5 mm | Blue | 100 pcs |



Packaging

Shipping Carton

100 pieces
Gross Weight 4.8 kg
Dimensions
326 x 198 x 190 mm



Regulatory Status

Medical Device Class I, non-sterile


Manufacturer

Kimetec GmbH (Germany)

Distributor

TERUMO EUROPE N.V.
Interleuvenlaan 40, 3001 Leuven, Belgium

Instruction for Use

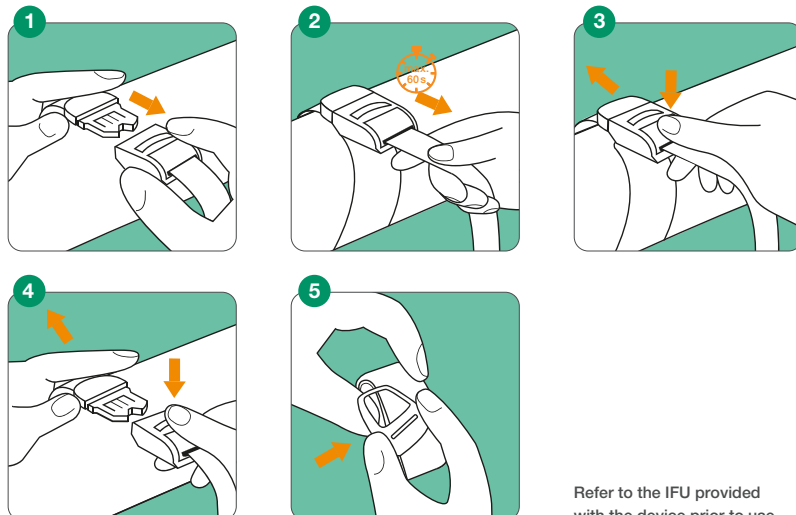

Refer to the IFU provided with the device prior to use.

Complete Range

Our tourniquet portfolio includes suitable tourniquets for patients of all ages.



Your Terumo partner

TERUMO EUROPE N.V.

Terumo Corporation Terumo Europe N.V.
+81 3 6742 8500 +32 16 38 12 11

www.terumo-europe.com

All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners. Published by Terumo Europe N.V.